




Trilogy[®] Acetabular System Liner Inserter

Assembly/Disassembly
Instructions

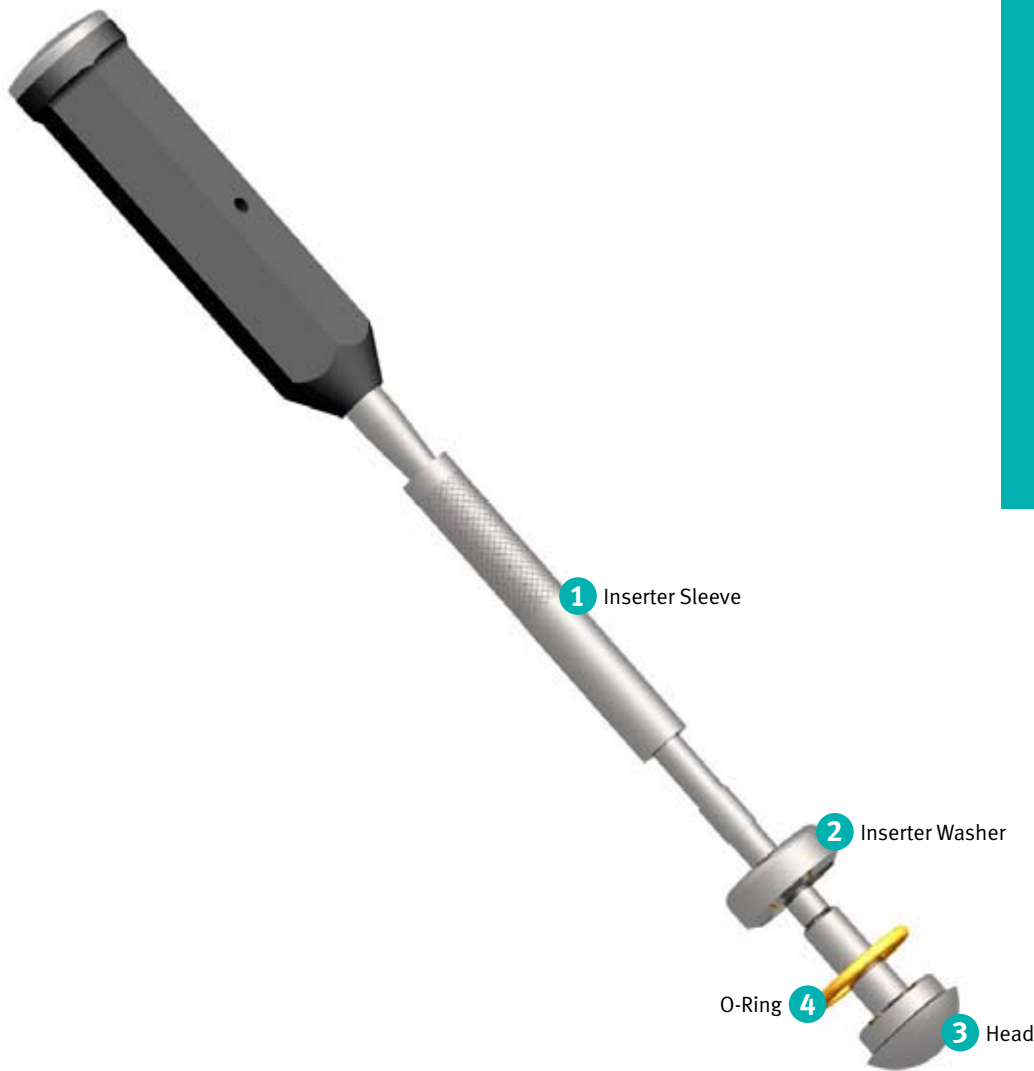

Disassembly Instructions

1. Loosen Liner Inserter Sleeve **1** threads by turning counter-clockwise until the threads are fully disengaged. Sleeve **1** should be loose on the shaft of the Inserter.
2. Pull the Liner Inserter Washer **2** straight out from the Head **3**. Washer **2** should be loose on the shaft of the Inserter.
3. Pull the rubber O-ring **4** straight out from the Head **3**. O-ring **4** should be loose on the shaft of the Inserter.

Note: The Liner Inserter must be completely disassembled for cleaning and steam sterilization.

Assembly Instructions

1. Pull O-ring **4** to the Head **3** until it is fully seated underneath it.
2. Push Washer **2** firmly up against O-ring **4**.
3. Push Sleeve **1** up against Washer **2**.
4. Secure the assembly by turning the Sleeve **1** clockwise until the threads are fully engaged.



0086

Zimmer, Inc.
1800 West Center Street
Warsaw, Indiana 46580
USA



Zimmer U.K. Ltd.
9 Lancaster Place
South Marston Park
Swindon, SN3 4FP, UK

©2007 Zimmer, Inc.



zimmer
Confidence in your hands[®]